


3M MASK



HOW DOES THE RESPIRATOR PROTECT US?

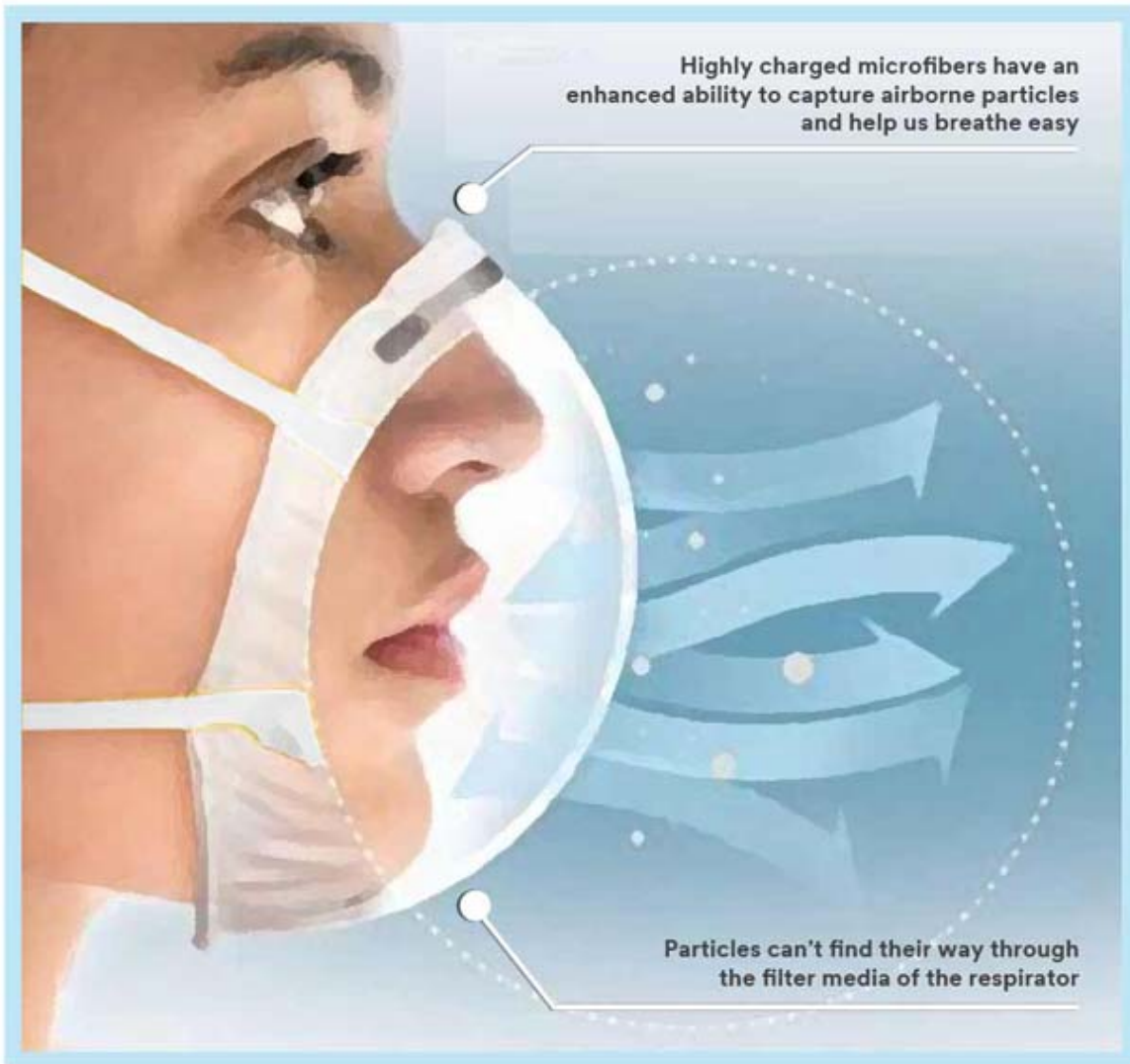

HIGHLY EFFECTIVE FILTER SYSTEM


High filtering efficiency $\geq 80\%$ for 0.3 Micron particle size,
and 100% filtering efficiency for 2.5 micron particle size.
Protects from fine dusts, Ashes, Pollen, smog, etc

Outer layer Material:


Non-woven material
3D design
Blocks dust

Filter layer Material:

3M electrostatic filter
Filters at least 80% of airborne particles
Seals and prevents leakage

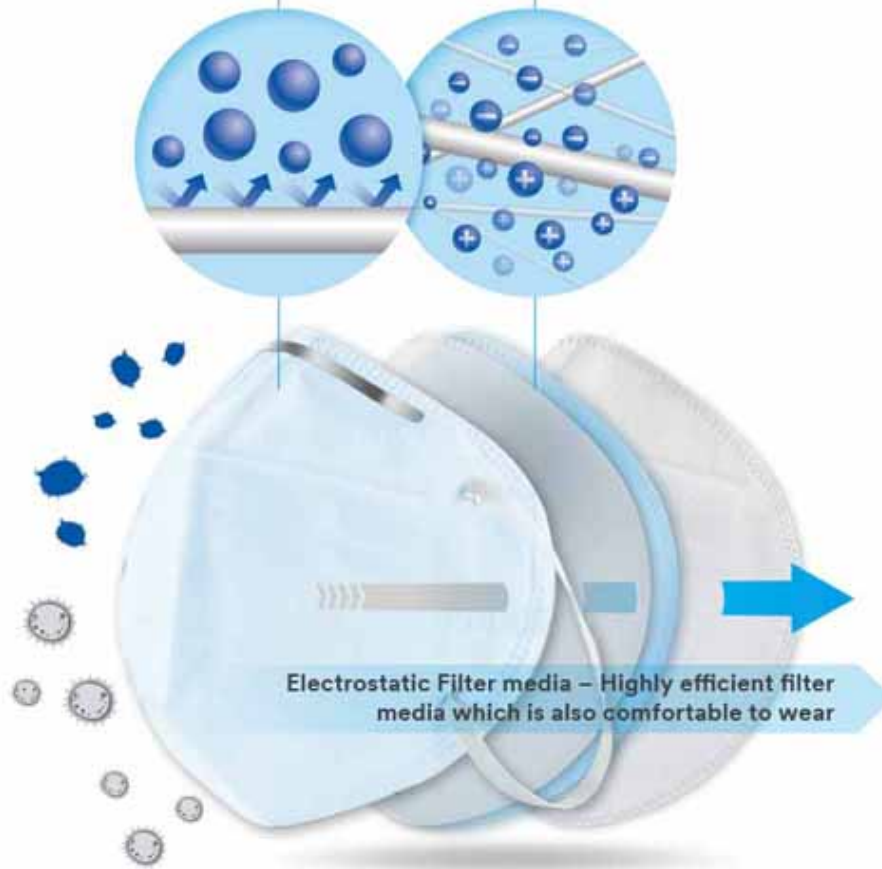

PRODUCT INFORMATION

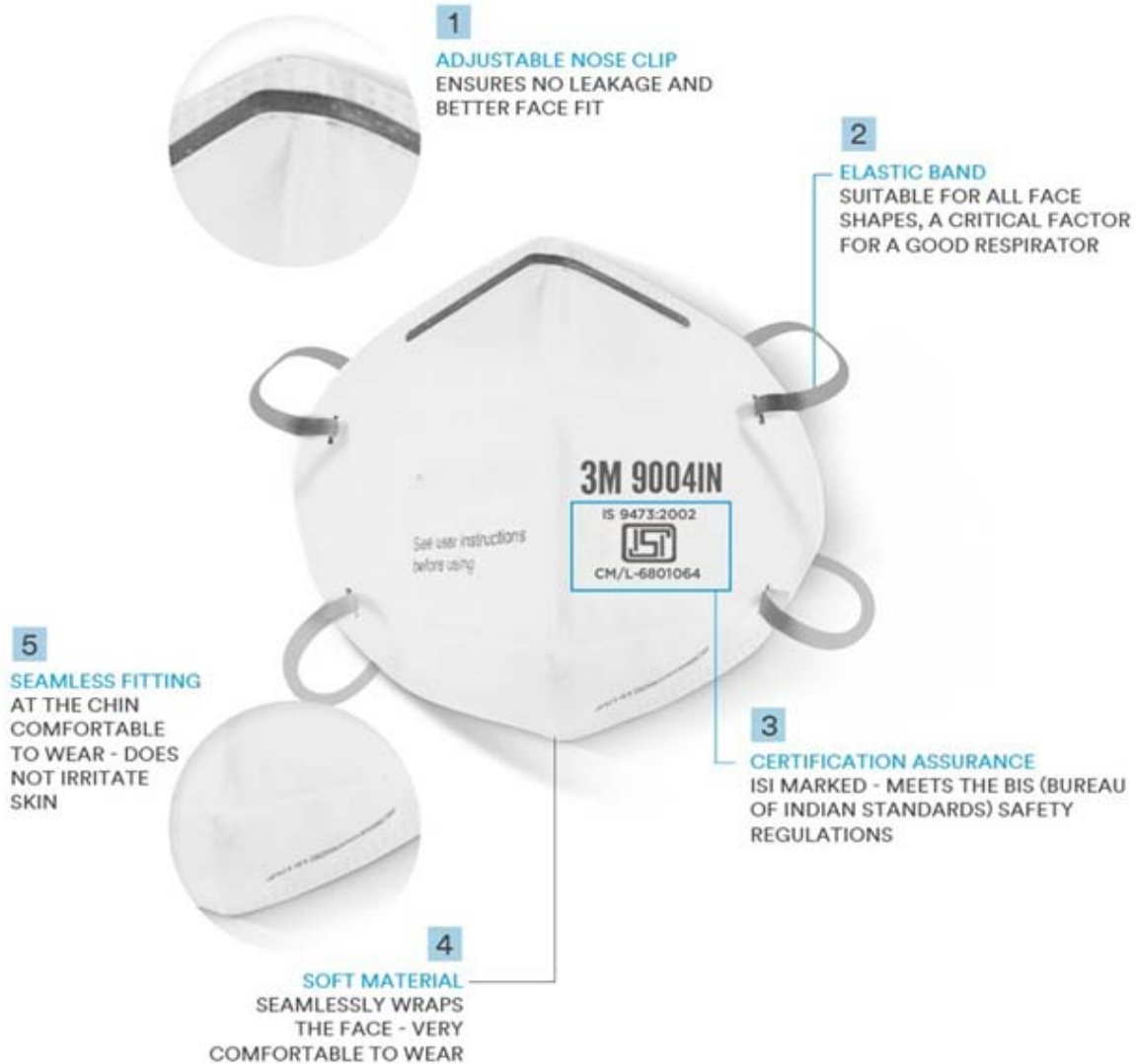

HOW TO WEAR



1

Place the respirator over your nose and mouth. Be sure the metal nose clip is on top.



2

Pull the top strap over your head until it rests on the crown of your head above your ears.



3

Pull the bottom strap over your head until it rests just below your ears.



4

Using both hands starting at the top, mold the metal nose clip around your nose to achieve a secure seal. Note: Pinching the nosepiece using one hand may result in improper fit and less effective respirator performance. Use two hands.